



SpiritCare Drainage Bag



Instructions for use

Product Name
Night Drainage Bag

Trading Name
SpiritCare Drainage Bag

Properties

The SpiritCare Drainage Bag has a large 2000ml urine drainage capacity and is made from latex-free medical grade PVC. It consists of a urine drainage bag and an easy to use T- valve outlet. The product is a Class I non-sterile medical device, intended for single use only. The front is transparent, and the back is white to ensure measurements are easy to read. The anti-reflux valve and anti-kink tubing reduces the risk of backflow and prevents accidental leakage.

Indications

This product is designed to collect urine drainage from a catheter. It is used to drain accumulated urine and the 120cm tube length makes it especially suitable for bedbound patients.

Contraindication and adverse effects

None

Measuring function

None. The meter display provides an estimation to aid medical staff but is not suitable for an accurate measurement. Please use a suitable graduated receiver for this.

Application

1. Always wash your hands before use and after emptying the bag.
2. Position the bag at or below the bladder level.
3. Observe periodically to ensure urine is flowing freely.

Safety instructions

1. For single use only. The product must not be reused.
2. Do not use, if packaging is opened or damaged, if the product is damaged or the expiry date is exceeded.
3. During use, the bag must hang upright to avoid looping and kinks in the tube.

Storage

Do not store in direct sunlight, at extreme temperatures or in high humidity.





Products available

	Code	Capacity	Tube Length
SpiritCare Drainage Bag	SH-CC022	2000ml	120cm

 Non-sterile	 Use by date
 Authorised representative in United Kingdom	 Conforms to 93/42/EEC
 Contains or presence of Phthalates	 Article no
 Latex-free	 Batch code
 Caution	 Consult the instructions for use
 Temperature limit	 Store in a dry place
 Do not reuse	 Keep away from sunlight
 Manufacturer	 Date of manufacture
 Medical device	 Unique device identifier
 Importer	

 Ningbo Greatcare Medical Instruments Co., Ltd.
No. 92 Cishanhe Rd., Chunxiao, Beilun, Ningbo,
Zhejiang 215830, China

 NPZ technology
Stirling House, Cambridge Innovation Park,
Denny End Road, Waterbeach, Cambridge, CB25 9QE, UK

 Distributed by: Spirit Healthcare Limited, Spirit House, Saffron Way, Leicester, LE2 6UP
Approval number: HEA4433APR23

