


# SpiritCare Incontinence Sheath

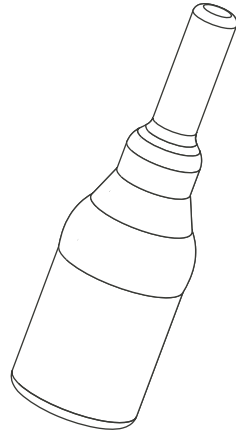

## Instructions for use

**Product Name**  
Incontinence Sheath

**Trading Name**  
SpiritCare Incontinence Sheath

### Properties

The SpiritCare Incontinence Sheath is a 100% silicone, latex-free, male external catheter or urinary sheath to help manage male incontinence in a discreet and secure way. It is a Class 1 non-sterile medical product. The medical adhesive on the inside of the sheath is skin-friendly providing safe adherence to the penis, patient comfort and minimises the potential for leakages. The one-piece system is easy to apply and is designed for easy removal. The anti-kink silicone funnel helps ensure continuous urine flow and can be securely connected to SpiritCare urine bags and all other major urine collection bags.

### Indications

The sheath is suitable for men with urinary incontinence.

### Contraindication and adverse effects

Allergic reaction to the adhesive can occur.

### Application

1. Always ensure that a man has been accurately measured before a sheath is selected.
2. Always wash your hands before starting.
3. Trim pubic hair from the base of the penis.
4. Wash and dry the penis.
5. Position over tip of penis and unroll along the full length of the penis.
6. When in place, press gently to create a seal and smooth to remove wrinkles.
7. Secure the urine bag to the patient's leg before connecting the tube to the extension of the sheath. This will help prevent twisting and will be more comfortable for the patient.
8. Ensuring the valve of the urine bag is closed, connect the tube at the end of the sheath to the urine bag.
9. To remove, use warm soapy water and gently unroll.

### Safety instructions

1. For single use only. Dispose of in the waste after use. The product must not be reused.
2. Do not use, if packaging is opened or damaged, if the product is damaged or the expiry date is exceeded.
3. Your healthcare professional will help select the correct size sheath.
4. Do not wear for longer than 24 hours.

### Storage

Store in a cool and dry place and keep away from sunlight.

## Products available

	Code	Size
<b>SpiritCare Incontinence Sheath STANDARD</b>	SH-CC007	24mm
	SH-CC008	28mm
	SH-CC009	31mm
	SH-CC010	35mm
	SH-CC011	40mm
<b>SpiritCare Incontinence Sheath SHORT</b>	SH-CC012	24mm
	SH-CC013	28mm
	SH-CC014	31mm
	SH-CC015	35mm
	SH-CC016	40mm
<b>SpiritCare Incontinence Sheath WIDE</b>	SH-CC017	24mm
	SH-CC018	28mm
	SH-CC019	31mm
	SH-CC020	35mm
	SH-CC021	40mm



Non-sterile



Wash hands



Authorised representative  
in the European Community



Contains or presence of Phthalates



Latex-free



Caution



Temperature limit



Do not reuse



Manufacturer



Date of manufacture



Use by date



Conforms to 93/42/EEC



Article no



Batch code



Consult the instructions for use



Store in a dry place



Keep away from sunlight