

SpiritCare Leg Bag

With T-Valve



Instructions for use

Product Name
Leg Bag

Trading Name
SpiritCare Leg Bag

Properties

The SpiritCare leg bag is composed of medical grade PVC film and consists of a urine bag with anti-kink discharge and drainage tube and an easy to use non-return T-valve to reduce leakages. This is a Class I (sterile) medical device. The adaptor enables connection to an additional urine bag if required.

Indications

The leg bag is used in the management of urinary incontinence to collect urine drained from a catheter or other medical devices.

Contraindication and adverse effects

None

Measurement function

None. The meter display provides an estimation to aid medical staff but is not suitable for an accurate measurement. Please use a suitable graduated receiver for this.

Application

1. Always wash your hands before and after emptying the bag.
2. Ensure the bag is fixed below the bladder
3. If the patient is in a lying position, place the leg bag vertically below the level of the bladder.
4. Close the drainage valve before you connect the bag to the catheter or another medical device.
5. Monitor the system at regular intervals to ensure the urine is flowing freely.
6. To empty, open the drainage valve and allow the urine to drain into the toilet or suitable container.
7. Close and clean the drainage valve. Clean with a disinfectant cloth and use this to remove all residue from the drainage valve.
8. When connecting the leg bag with a urine bag make sure that the connection is airtight.

Safety instructions

1. Dispose of used bags in your usual household collection bins.
2. This product should only be used under the supervision of trained clinicians.
3. Do not use if packaging is opened or damaged, if the product is damaged or the expiry date is exceeded.
4. During use, ensure there are no kinks or obstructions in the tube.

Storage

Do not store in direct sunlight, at extreme temperatures or at high humidity.


Products available

	Code	Capacity	Tube Length
SpiritCare Leg Bag - Short	SH-CC001	350ml	10cm
	SH-CC003	500ml	10cm
	SH-CC005	750ml	10cm
SpiritCare Leg Bag - Long	SH-CC002	350ml	30cm
	SH-CC004	500ml	30cm
	SH-CC006	750ml	30cm

 Sterilised using ethylene oxide

 Do not re-sterilise

 Authorised representative in the European Community

 Contains or presence of Phthalates


 Latex-free

 Caution


 Temperature limit

 Do not reuse

 Manufacturer


 Date of manufacture


 Use by date


 Conforms to 93/42/EEC


 Article no

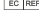
 Batch code

 Consult the instructions for use

 Store in a dry place

 Keep away from sunlight

 Ningbo Greatcare Trading Co., Ltd.
Unit 93, Building 12, No. 818, Qiming Road,
Yinzhou 315105 Ningbo, Zhejiang, China

 Greatcare Medical GmbH
Bonner Str. 31, 50389 Wesseling, Germany
Made in China

Distributed by: Spirit Healthcare Limited, Spirit House, Saffron Way, Leicester, LE2 6UP
Approval number: HEA3707FEB21